Making Your Life Better Every Day
The true pouch cleanser that can improve your life.

INSTRUCTIONS FOR USE

- 1** Unpack OSTI-KLENZ™ and official bottle.
- 2** Screw OSTI-KLENZ™ to bottle filled with clean water (figure 1)
- 3** While sitting on a commode, hold pouch facing you and open drainable end by removing clamp or opening the velcro closing. Insert OSTI-KLENZ™ with open end face down into pouch 3 inches (just past the neck) (figure 2)
- 4** Hold pouch facing you and squeeze bottle until contents are emptied into pouch (figure 3)
- 5** Remove OSTI-KLENZ™ by turning it 180 degrees with end facing upward and remove completely from pouch
- 6** Fold up base of pouch 3 or 4 times or apply clamp to seal water and waste in the pouch. With both hands gently move waste and flush about in pouch.
- 7** With one hand on the upper part of pouch and the other on the lower opening, simultaneously press down quickly with the hand on the upper part of the pouch and the other hand on the lower opening. Release and flush will flow out of the bag with pressure and thus clean the pouch thoroughly. If necessary, repeat process immediately with second bottle.
- 8** When pouch is empty, use a dry tissue to blot end and fold pouch into original position. Then apply clamp or attach velcro.
- 9** Wash OSTI-KLENZ™ thoroughly. Refill bottle and screw on OSTI-KLENZ™. You are ready for the next cleansing.
- 10** Use detergent and water regularly to clean OSTI-KLENZ™.

Note: Tip of OSTI-KLENZ™ is always 6 inches from the Stoma.

figure 1

figure 2

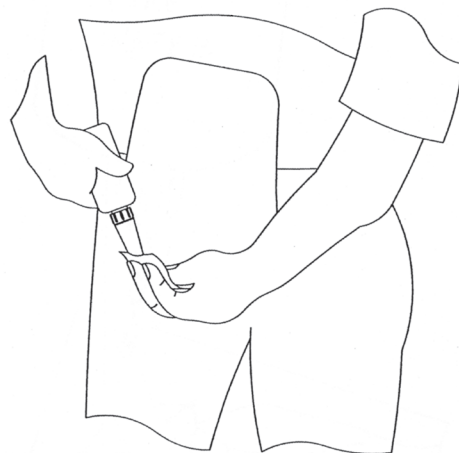
figure 3

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